

What is ASSURANCE AB™?

The ASSURANCE AB™ COVID-19 IgM/IgG Rapid Antibody Test is a rapid test for the qualitative detection of IgM and IgG antibodies associated with the SARS-CoV-2 virus. ASSURANCE AB™ is a single-use, self-contained test that provides results in 15 minutes, helping authorities, employers and healthcare workers make informed decisions to contain the spread of COVID-19.

Who is ASSURANCE AB™ for?

ASSURANCE AB™ is intended for individuals suspected of COVID-19 infection by a healthcare provider or who were exposed to an individual infected with COVID-19.

ASSURANCE AB™ is for use by laboratories and healthcare workers at the point-of-care.

What is antibody testing?

Antibody assays have the ability to detect past infections and indicate probable immunity.

ASSURANCE AB™ detects the presence of antibodies IgM and IgG, developed in response to the SARS-CoV-2 virus.

- ▶ IgM antibodies: develop EARLY after SARS-CoV-2 virus exposure
- ▶ IgG antibodies: develop LATER after SARS-CoV-2 virus exposure; indicates longer-term immunity


ASSURANCE AB™ highlights

- ▶ Results in 15 minutes
- ▶ NO laboratory equipment required, easy-to-use
- ▶ Reduces risk of spreading the virus to healthcare workers administering tests or technicians processing tests
- ▶ Provides actionable information to all patients to help determine if a COVID-19 viral PCR test is needed
- ▶ Provides insightful information for when, after consultation with a healthcare provider, it is safe for individuals to go back to work and resume normal activities of daily life
- ▶ Testing accuracy is high due to antibodies being more uniformly available in blood samples than virus in nasopharyngeal swabs

ASSURANCE AB™ test administration

ASSURANCE AB™ requires a small sample of the patient's blood, usually collected via fingerstick. Blood and diluent are dispensed onto the self-contained testing cassette. Once applied, results appear on the testing device within 15 minutes. Results indicate the presence of antibodies, developed in response to the SARS-CoV-2 virus associated with the COVID-19 pandemic. Testing cassettes are embedded with a control sample.

Understanding ASSURANCE AB™ results

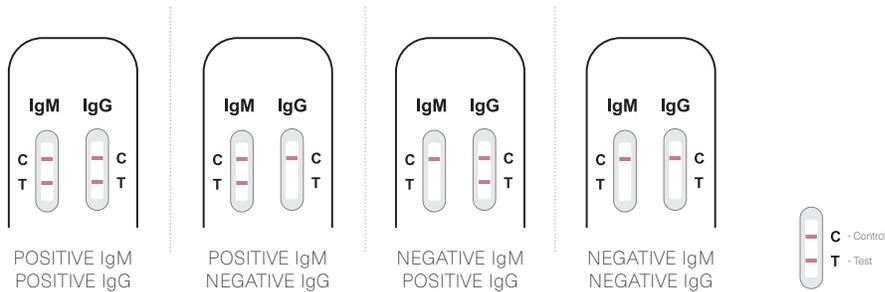

If either or both of the control lines do not develop, the assay is invalid

What does a positive test result mean?

A positive test result indicates that the patient has been exposed to the SARS-CoV-2 virus and has developed antibodies associated with immunity. A patient with a positive test result may or may not be contagious. Laboratory test results should always be considered in the context of clinical observations and epidemiological data in making a final diagnosis and patient management decisions. Please refer to Instructions For Use.

What does a negative test result mean?

A negative test result indicates that the patient has not developed detectable antibodies associated with the SARS-CoV-2 virus and most likely does not have immunity to the virus. A person with a negative test result may or may not be contagious. Please refer to Instructions For Use.

About Predictive Laboratories, Inc.

Predictive Laboratories' molecular and genetic diagnostics focus on hard-to-diagnose and hard-to-detect diseases. Leveraging its vast genetic and genomic library, Predictive offers earlier detection of a variety of diseases through genetic assessments to guide personalized precision medicine.

Equipped with a state-of-the-art CAP and CLIA accredited laboratory, Predictive operations perform next-generation sequencing experiments, including whole exome sequencing, gene and genetic marker panels, and low-pass whole genome analysis of embryos for aneuploidies. Predictive arms physicians with the most robust diagnostic tools to provide personalized precision treatment for their patients. For more information, visit www.predictivelabs.com.

Contact Information:

Predictive Laboratories
info@predictivelabs.com
 855-497-3636